



C50Y100



C100M30



K70

正

21\*28.5cm

反 panyangjuan2019.12.9

# BURN CARE RELIEVE BURN GEL



**PRODUCT NAME:** Relieve Burn Gel  
**GENERAL NAME:** Sterile Burn Gel

## ► SPECIFICATION

Specification	Size/ Volume
Gel Sachet	Volume:0.5~125g
Gel spray bottle	Volume:50~125g
Dressing	Volume:2~750g Size: (2.5cm~40cm)x(5cm~60cm)

## ► PRODUCTS PERFORMANCE

Relieve Burn Gel is a water-based, water-soluble gel that draws the heat out of a burn while it relieves the pain, cools the skin and protects against airborne contamination. By applying Relieve Burn Gel as soon as possible, the heat of the burn is dissipated, helping to prevent the heat from progressing deeper into the skin. The gel consists of 90% purified water and has the same cooling effects as water, but because it is thick, it stays on the burn and evaporates at a very slow rate. Relieve Burn Gel dose not interfere with medical attention at hospital, such as autograft etc. Relieve Burn Gel is totally water-soluble. Any residual Relieve Burn Gel on a burn wound will be removed with the first bath by doctor or nurse. Even when Relieve Burn Gel is used for the definitive treatment of a small burn, it is removed with each washing. Relieve Burn Gel is sterile by Gamma and have a five-year shelf life .

## ► INTENDED USE

Relieve Burn Gel are mainly used for covering the wounds of burn, to keep the wounds moist. The product is emergency used for 1st degree and 2nd degree burn wounds. The product draws the heat out of a burn while it relieves the pain, cools the skin and protects against airborne contamination.

## ► USAGE METHOD

### I. DRESSING

For heat burn  
Tear open Burncare Sachet and lay the dressing across the wound site.  
Squeeze the remaining gel from the sachet across the dressing. Allow to remain in contact with the wound site for at least 30mins.  
If necessary, secure Burncare in place with a conforming bandage, taking care not to apply too much pressure.  
Seek Medical attention if burn is severe or covers a large area.  
Burncare gel is soluble and can be rinsed off by saline or sterile water if necessary.

For chemical burns  
If it is liquid chemical it must be irrigated off site by cold water for approximately 15~20mins.  
If it is powder chemical it must be brushed off site, then wash the wound site by cold water for approximately 15~20mins.  
Follow the instruction as per heat burn.

### II. Gel:

For heat burn  
Open Burncare package, apply gel across the wound site, form a generous layer to affected area. Do not rub the wound site. Allow to remain in contact with the wound site at least for 30mins. Seek Medical attention if burn is severe or covers a large area.  
Burncare gel is soluble and can be rinsed off by saline or sterile water if necessary.

For chemical burns  
If it is liquid chemical it must be irrigated off site by cold water for approximately 15~20mins. If it is powder chemical it must be brushed off site, then wash the wound site by cold water for approximately 15~20mins.  
Follow the instruction as per heat burn.

## ► WARNINGS:

1. Emergency first aid for burns, scalds, sunburn only. Do not use for healing.
2. Single use only. Do not use if package is opened or damaged.
3. Do not rub the gel into affected area.
4. Discontinue if irritation occurs.
5. Seek Medical attention immediately if burn is severe or covers a large area.

## ► CAUTION

1. This product is sterile products, cannot be used when the package is damaged.
2. Disposable products, cannot be reused. Repeated using can cause infection.

## ► STORAGE CONDITIONS

This product should be stored in the relative humidity of not more than 80%, the temperature is not more than 40°C, no other toxic substances, well-ventilated room.

②	Do not re-use
Ⓢ	Do not use if package is damaged
STERILE R	Sterilized using irradiation
📅	Use by :year and month
LOT	Batch number
⚠	Attention, see instructions for use
🔄	Do not re-sterilize
📖	Consult instructions for use
🏭	Manufacturer
EC REP	Authorised representative in the european community

Roosin Medical Co., Ltd.  
No.8 Yuandong Road, Kouan Town, Gaogang,  
Taizhou City, Jiangsu Province 225321, P.R.China  
TEL:+86(523)86908085 FAX: +86(523)86908085

Prolinx GmbH  
Brehmstr. 56, 40239, Duesseldorf  
Germany  
TEL: 0049 211 3105 4698 FAX: 0049 2131 4051968-9