


Advanced

CHEST SEAL VENTED

DESCRIPTION

RHINO RESCUE introduces the newest product in our line of hydrogel chest seal, Advanced Chest Seal Vented. First, the double-layer design significantly improves the sealing performance, and air cannot enter the body from the wound; Second, the unique ultrasonic welding design can effectively reduce the return of body fluids.

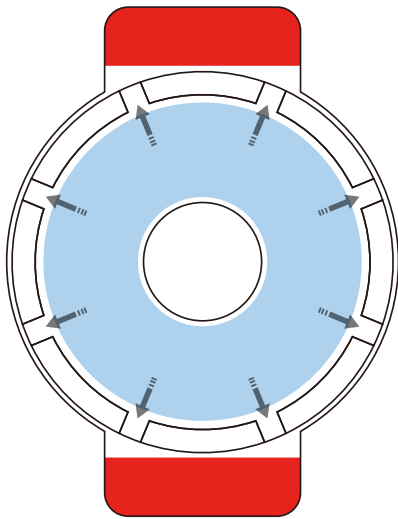

* The blue area is both a drainage channel and a storage space, which plays a vital role in the case of a large amount of drainage

FEATURE

- **DOUBLE-LAYER DESIGN:**
Improve the sealing performance, and air cannot enter the body from the wound.
- **UNIQUE ULTRASONIC WELDING:**
Effectively reduce the return of body fluids.
- **Double handle design:**
Configured for ease of use in low-light environments.


SPECIFICATION

PRODUCT SIZE: 17×11cm/6.7×4.3in


PRODUCT WEIGHT: 50g

PRODUCT MATERIAL: Hydrocollold


INSTRUCTIONS FOR USE

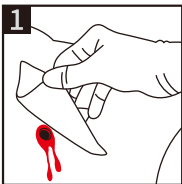


1 Clean and dry wound.

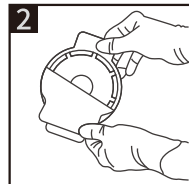

2 Remove protective liner from adhesive coated surface.

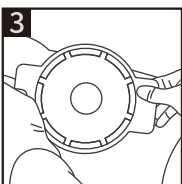

3 Place it on patient's wound, adhesive side down.

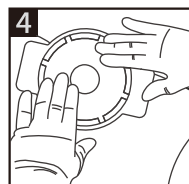

4 Press it firmly to skin to assure an occlusive seal.

