


Rhino Rescue


TOURNIQUET ONE-HANDED APPLICATION




01
REMOVE tourniquet from the first aid kit.

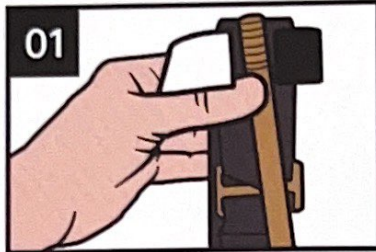


02
INSERT the wounded extremity through the loop of the self-adhering band.

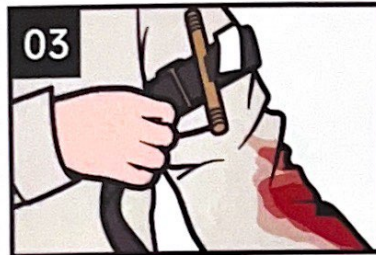


03
POSITION the tourniquet above the bleeding site, high on the extremity over the clothing/uniform.

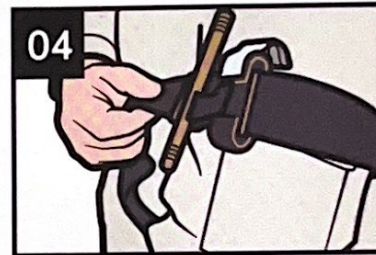


04
SECURE elastic bandage tails and knot using tape, prevent the knots from unraveling.

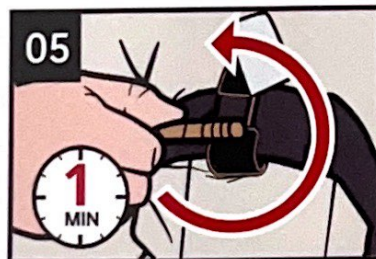


05
TWIST the rod until bleeding has stopped (complete steps 1-5 in under 1 min).

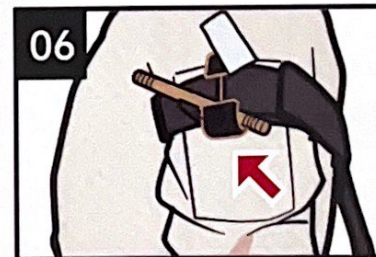

06
LOCK the windlass rod in place with the windlass clip.

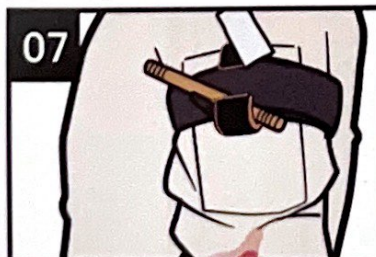

07
ROUTE the self-adhering band around the rod and between the clips.

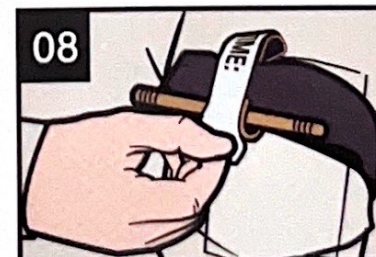

08
SECURE with the windlass safety strap.

TOURNIQUET